# GLUMA® Etch 35 Gel

(GB)	Instructions for use .....	3
(ES)	Instrucciones de uso .....	7
(GR)	Οδηγίες χρήσης .....	11
(CZ)	Návod k použití .....	15
(HU)	Használati utasítás .....	19
(LV)	Lietošanas instrukcija .....	23
(LT)	Naudojimo instrukcija .....	27
(PL)	Instrukcja obsługi .....	31
(HR)	Upute za uporabu .....	35
(RO)	Instrucțiuni de utilizare .....	39
(TR)	Kullanma talimatı .....	43
(RU)	Инструкция по применению .....	47
(AE)	تعليمات الاستخدام .....	51
(CN)	使用说明 .....	55
(KR)	사용 설명서 .....	58
(SK)	Návod na použitie .....	62
(BG)	Инструкции за употреба .....	66
(UA)	Інструкція із застосування .....	70
(EE)	Kasutusjuhend .....	74
(SI)	Navodila za uporabo .....	78
(MK)	Упатство за употреба .....	82
(RS)	Uputstva za upotrebu .....	86

**KULZER**  
MITSUI CHEMICALS GROUP

Pasty

Pastoso

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Pastovité

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Pastveida

Pastos pavidalo

W postaci pasty

U obliku paste

Pastā

Macun gibi

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Pastozen

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U obliku paste

**Medical Device – for use by dental health care professionals only.**

### **Intended purpose**

GLUMA Etch 35 Gel is an etching agent based on orthophosphoric acid. It is used to etch enamel and condition dentine in the context of adhesive filling therapy.

### **Indications**

Enamel etching and enamel/dentine etching in the context of the total etch technique prior to the adhesive bonding of:

- Composite and compomer restorations
- Laboratory-fabricated, indirect restorations (e.g. inlays, crowns, bridges, veneers)
- Sealants

### **Contraindications**

Application to dentine close to the pulp (less than 1 mm). In case of known or suspected allergy to components of the product, its use is contraindicated.

### **Patient target group**

This dental material is suitable for patients requiring dental treatment for the following indications with consideration of the contraindications. For pregnant and nursing women, the treatment risks must be weighed carefully against the benefits taking into consideration the unborn child or infant.

### **Intended user**

For use by dental health care professionals only.

### **Clinical benefit**

Kulzer medical devices ensure the rehabilitation of oral functions such as chewing, speaking and aesthetics. They stabilize the remaining dentition and/or the alveolar ridge restoratively or prosthetically.

### **Potential side effects**

Hypersensitivities to the product or its components cannot be excluded in individual cases.

### **Composition**

GLUMA Etch 35 Gel contains: water, phosphoric acid, silicon dioxide.

### **Product application**

Prior to treatment, carefully clean tooth with fluoride-free polishing paste. The use of a rubber dam is recommended. Visually inspect for damage before use. Damaged products must not be used.

1. After preparation in accordance with adhesive filling therapy, rinse the cavity and dry with oil-free air.
2. Starting from the bevelled enamel areas, apply GLUMA Etch 35 to the entire cavity surface, including the dentine. Ensure that the enamel is conditioned for at least 20 s, a maximum of 30 s and the dentine for 15 s. If desired, only apply GLUMA Etch 35 to the enamel and let set for 20 – 30 s. Etch unprepared enamel, e.g. in case of fissure sealing, for at least 30 s and no more than 60 s. In case of fluoridated enamel, etching for no more than 60 s may also be required.

**Caution:** In case of deep cavities, we recommend covering the dentine near the pulp with a calcium hydroxide preparation and a base (e.g. glass ionomer cement) prior to etching.

3. Use a water sprayer to completely rinse off GLUMA Etch 35 for 20 s under suction. The cavity must be completely free of any remnants.

4. Dry the cavity with oil-free air (do not dry out dentine!). The etched enamel surface should appear chalky white. In case of contamination of the tooth surface (e.g. with saliva), the etching must be repeated (for a maximum of 10 s dwell time).
5. Perform further treatment with adhesive (e.g. GLUMA Bondings) and composite (e.g. CHARISMA®) in accordance with manufacturer directions.

GLUMA Etch 35 Gel may be applied using a brush or cannula. Both brushes and cannulas are for one-time use only.

Release the plunger after applying the gel. When using syringes in mouth, use common hygienic protective sleeves to prevent contamination. Do not reuse after contamination. After use, close GLUMA Etch 35 Gel with the appropriate plug or cap.

### **Interactions**

Liner and base materials, such as calcium hydroxide, may be dissolved by the acid, depending on their type and composition.

### **General information**

Please pay attention to the following safety instructions and all processing instructions in other sections. This product must be used according to the instruction for use only. Any use not complying with these instructions is at the discretion and sole responsibility of the dental professional itself.

### **Warnings and safety instructions**

GLUMA Etch 35 Gel contains 35 % phosphoric acid and can cause chemical burns. May cause skin and eye irritation. Avoid contact with skin. Skin contact may cause sensitization. Wear suitable protective gloves. In case of contact with skin, immediately wash with plenty of water and soap. Avoid contact with eyes. Eye

protection is recommended for the patient. In case of contact with eyes, rinse with water for several minutes. If symptoms persist, consult an eye specialist.

The etching agent must not be swallowed by the patient, otherwise incompatibilities may occur.

### **Special advice**

According to EU Medical Device Regulation users/ patients are obliged to report serious incidents with a medical device to the manufacturer and to the competent authority of the country, the incident occurred.

### **Storage conditions**

Store below 25 °C (77 °F). Protect from sunlight. Store in a dry place. Do not use after expiry date.

### **Disposal information**

Recommendation: Dispose of in accordance with official regulations. Do not dispose of contents or partially emptied packaging in the household waste or allow them to enter the sewage system.

European Waste Catalogue: 180106 Chemicals consisting of or containing hazardous substances.

### **Complaint notice**

Please quote batch number and article number or UDI in all correspondence about the product.

Article number **REF** and batch number **LOT** or UDI: See note on packaging.

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Kulzer GmbH