SPARK™

INSTRUCTIONS

for Use and Care


SPARK
SPARK
TRUCOLOR

North America

INTENTIONALLY BLANK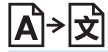

INDEX

English	4
Français (French)	15

Indications For Use

The Spark headlight is an illumination device attached to spectacles, worn as a visual aid, intended to illuminate oral structures and operating areas.

Intended Purpose: This device is intended to illuminate oral structures and operating areas.

Intended Users: This device is intended to be used by qualified medical and healthcare professionals including, but not limited to, dentists, dental hygienists, medical doctors and medical professionals.

Intended Patient Population: This device is intended for use on patients during general dental and surgical procedures.

Clinical Benefits: When attached to loupes or other eyewear, this device illuminates the field of view of the wearer to improve visibility during procedures.

Precautions

1. Before use, ensure all screws and removable components are secured to the device.
2. To prolong the life of the batteries, do not store in a fully charged or discharged state.
3. Only use the headlight that ships with the device.

DO NOT

- Do not disassemble or modify the device
- Do not remove the product serial number labeling
- Do not begin a procedure while the low-battery indicator light is illuminated or flashing
- Do not hang or pull on exposed cables
- Do not tightly bend or wrap the headlight cable
- Do not spray directly onto the device
- Do not run any part of the device under water or submerge in any liquid or an ultrasonic cleaner
- Do not use an autoclave or chemiclave, glutaraldehyde, iodophor, or any other sterilization method
- Do not use a cleaner with alcohol concentration greater than 70%

Contraindications None known.

Adverse Reactions None known.

Disposal Instructions

When you plan to dispose of the device, please return to Orascoptic for recycling. The European Union Waste Electrical and Electronic Equipment (WEEE) directive forbids the disposal of electrical and electronic equipment as unsorted municipal waste, requiring that it be collected and recycled or disposed of separately. Hazardous substances are present in electrical and electronic equipment and present potential risk to human health and environment if disposed of in municipal landfills which are not designed to prevent migration of substances into the soil and groundwater. If you need assistance when disposing the device, please call one of our Customer Care Specialists or your authorized Orascoptic distributor.

This device contains lithium-ion cells. Recycle or dispose of the batteries according to national, state, and local regulations.

Transport and Storage

Warnings

1. The device must be used exclusively for the intended purpose described.
2. Do not use the device if any component is damaged, loose, detached, or missing. This includes battery packs, mounting clip, headlight, and electrical contacts/plugs. Parts that are broken, missing, distorted or worn should be replaced immediately. Should such repair or replacement become necessary, ORASCOPTIC OR AN AUTHORIZED SERVICE DEALER SHOULD PERFORM ALL REPAIRS TO THIS PRODUCT.
3. The use of power adapters and cables other than those provided for use with the device may result in increased emissions or decreased immunity and may cause the system to be non-compliant with the requirements of IEC 60601-1-2. Only the power adapter and cable that ships with the device should be used during charging.
4. This product maintains compliance with blue light testing standards for safety set by the International Electrotechnical Commission (IEC). As with all lighting products, there are risks to staring directly into the light.
 - IEC 62471 Risk Group 2 - CAUTION: Possibly hazardous optical radiation is emitted from this product. Do not stare at operating lamp. May be harmful to the eyes.
5. Misuse of the batteries may cause them to become hot, explode, or ignite and may cause serious injury. Be sure that the following safety warnings are expressly followed:
 - Never open or modify the battery pack, mounting clip or power adapter.
 - Do not use outside.
 - Protect from fluids or damp environments.
 - Maintain operational environment of 32° to 77° F (0° to 25° C).
 - Do not short circuit the electrical contacts.
 - Do not bring any part of the device into contact with fire or severe heat over 158°F (70°C)
 - If you determine that the battery is hot, emits an odor, or changes color or shape, disconnect power immediately. The battery must not be used again.
6. Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.
7. Portable radio frequency communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 12 inches (30 cm) to any part of this device, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.
8. ADVERSE EVENTS: If a serious incident occurs with this medical device, report it to the legal manufacturer, Metrex Research, LLC dba Orascoptic, or the authorized representative, Kerr Italia, S.r.l., and to the competent authority for the country in which the user/patient is established.

Limitations of Use None known.

Getting Started With Your New Spark Headlight

Universal Mount

The Spark headlight (Universal Mount) can attach to a majority of loupes and other eyewear.

The Spark headlight universal system includes these main components:

Orascoptic through-the-lens (TTL) Mount

The Spark headlight (Orascoptic through-the-lens "TTL" Mount) is a lighter-weight headlight that attaches only to Orascoptic-branded loupes.

The Spark headlight through-the-lens system includes these main components:

Accessories Included

USB-to-Micro USB Cable, Power Adapter, Optical Screwdriver

Please call one of our Customer Care Specialists or your authorized Orascoptic distributor to order additional or replacement parts.

Cleaning and Disinfecting

Your Spark headlight is a precision electrical instrument manufactured from high quality components. With proper care, it will provide a long and useful life. You must adhere to the following cleaning and disinfecting instructions or damage may result. Additionally, it is recommended to follow all instructions provided with your chosen cleaning/disinfecting solution.

Recommended Disinfectants

CaviCide™, CaviCide1™, CaviWipes™, or CaviWipes1™

NOTE These cleaning and disinfecting instructions only apply to your Spark series headlight and Orasoptic-branded loupe. If your headlight is attached to a non-Orasoptic branded product, follow that product's cleaning and disinfecting instructions separately.

1. Dampen or spray a soft cloth with your chosen cleaning/disinfecting solution, or use a disinfectant wipe. Squeeze excess fluid out of the cloth or wipe before using.
2. Hold the device securely while cleaning all surfaces with the damp cloth or wipe.
3. Discard dirty cloths or wipes and repeat the steps above until the device is clean.
4. If using a disinfectant, allow all surfaces to remain saturated for the proper duration (following instructions provided with your chosen disinfecting solution).
5. Use a clean and dry soft cloth to wipe all remaining cleaning/disinfecting solution from the surface of your device.
6. If your Spark headlight is attached to an Orasoptic-branded loupe, dampen a lens cleaning cloth with lens cleaning solution and wipe the inside and outside of the lenses, telescopes, and side shields to remove any streaks left from the cleaning and disinfecting process. One lens cleaning cloth is provided with all Orasoptic loupe products. Please call one of our Customer Care Specialists or your authorized Orasoptic distributor to purchase additional lens cleaning cloths.

DO NOT


- ... spray directly onto the device
- ... hang or pull on exposed cables
- ... use paper towel to clean lenses
- ... use a cleaning/disinfecting solution with alcohol concentration greater than 70%
- ... run any part of the device under water or submerge in any liquid or ultrasonic cleaner
- ... use an autoclave, chemiclave, glutaraldehyde, iodophor, or any other sterilization method

Mounting Your Spark Headlight (Universal) to a Loupe or Other Eyewear

The Spark headlight is secured to a loupe or other eyewear frame by tightening the mounting screw to clamp your frame between the mounting clip and mounting arms of the device. Access to the mounting screw is located behind the headlight bezel.

1. Tilt the headlight bezel down and use an optical screwdriver to loosen the mounting screw to open the mounting arms.
2. Push the device over the top of your loupe frame with mounting clip in front of the lenses and mounting arms behind the lenses.
3. Center the device over the bridge of your loupe frame and tighten the mounting screw until it is secure.
4. If the light is not centered in your field of view, you may need to loosen and adjust the mounting clip left or right as needed. Once centered to your field of view, retighten the mounting screw until the device is secure.


⚠ DO NOT attempt to remove your Spark headlight from the frame without first loosening the mounting screw, or damage may result.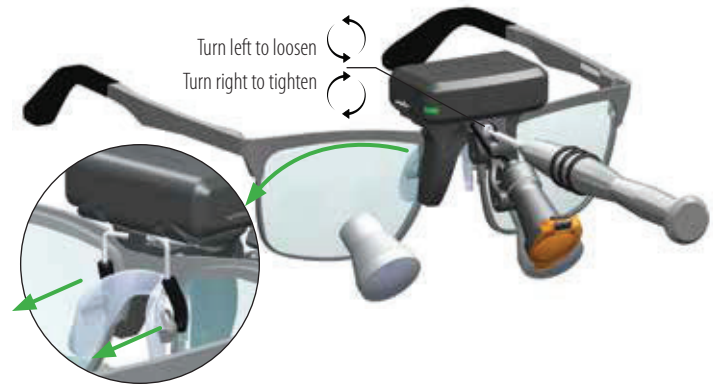

Mounting Your Spark Headlight (through-the-lens) to an Orascoptic Loupe

The Spark headlight is secured to an Orascoptic-branded loupe by tightening the mounting screws directly into the frame bridge.

1. If your Orascoptic loupe has a badge on the frame bridge, carefully remove it to reveal two threaded mounting holes.
2. Push two mounting screws through the mounting clip and use an optical screwdriver to slightly tighten both screws into the frame.
3. Center the device over the bridge of your loupe frame and tighten the mounting screws until it is secure.
4. If the light is not centered in your field of vision, you may need to loosen and adjust the mounting clip left or right as needed. Once centered in your field of view, retighten the mounting screws until the device is secure.

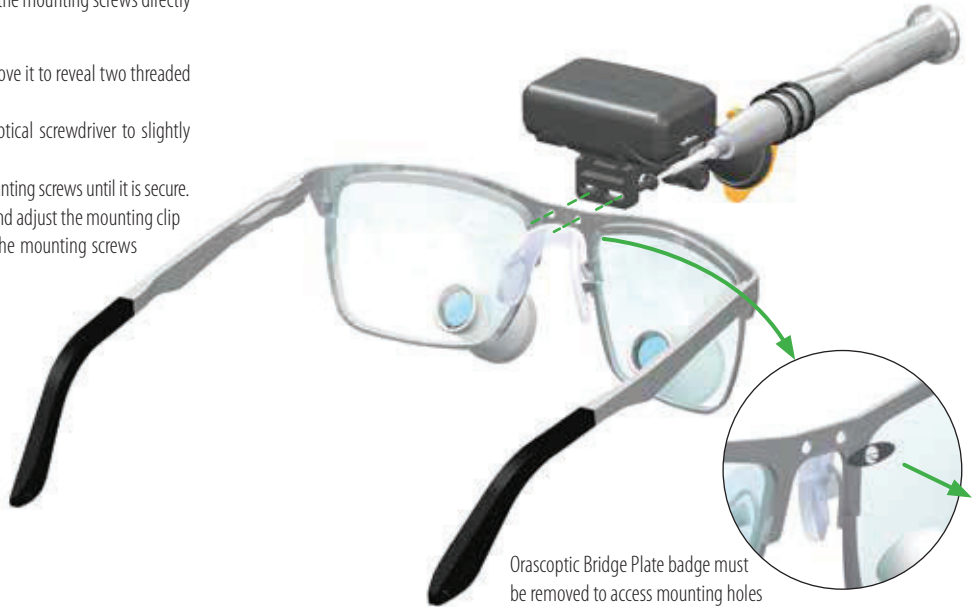Operating Your Spark Headlight

Charging the Battery Pack

Your Spark headlight comes with two battery packs so that one can be charging while the other is in use. Both battery packs are partially charged when shipped and should be fully charged prior to use. To charge a battery, plug the power adapter into an electrical outlet. Plug the USB-to-Micro USB cable into the power adapter and into the charging port on the back of the battery pack.

- The battery indicator light will blink **SCARLET** when the battery pack is nearly depleted and needs to be charged.
- The battery indicator light will illuminate **AMBER** while the battery pack is charging.
- The battery indicator light will illuminate **GREEN** when the battery pack is fully charged.

Installing the Battery Pack

The battery pack is secured to the mounting clip with locking tabs on each side. Depress both locking tabs when installing or removing the battery pack. The locking tabs will click into place when the battery pack is secure. After installing a battery pack, allow three seconds for it to activate before attempting to turn the headlight on.

Operating the Headlight

Your Spark headlight uses capacitive touch controls, making the light simple to operate during clinical procedures.

1. Install a battery pack onto the mounting clip.
2. Place your loupe and headlight on your head and adjust to a comfortable working position.
3. Touch anywhere on the front or side beveled areas of the battery pack to turn the headlight on and off.
4. Tilt the headlight to the appropriate angle to illuminate your visual field.

⚠ Do not point the headlight directly in your eyes or the eyes of others around you.

Using the Curing Filter

The curing filter should be used when working with light-cured composites. When applying these materials, unfiltered light from the headlight can cause premature curing of the composite. The curing filter is made up of two components, the black bezel ring that is secured to the end of the headlight and the orange filter that can be flipped up and down as needed during procedures. The curing filter can be removed from the headlight bezel if you don't require it for your practice.

- To remove the curing filter, grasp the headlight with one hand and the black bezel ring with another and pull. It is held with a friction fit and will separate from the headlight with moderate force.
- To attach the curing filter, flip the orange filter to the UP-position and carefully press the black bezel ring onto the headlight, making sure the hinge is at the top.

⚠ Do not directly pull or press on the orange filter.

Lithium-Ion Battery Technology

Rechargeable lithium-ion battery technology is ideal for portable devices like the Spark headlight.

- Do not leave the batteries unused for extended periods of time, either in the device or in storage.
- Rechargeable lithium-ion batteries have a limited life and will gradually lose their capacity to hold a charge, reducing the runtime of powered devices.
- If you find yourself charging batteries more often, or your device is not running as long as it should, please call our Customer Care Specialists or your authorized Orascoptic distributor to purchase replacement batteries.



INSTRUCTIONS FOR USE AND CARE

Orascoptic™ Spark™ Series Headlight

Symbol Glossary

Symbol	Symbol Title	Explanatory Text
	Medical Device	Indicates the medical device manufacturer.
	Catalogue Number	Indicates the manufacturer's catalogue number so that the medical device can be identified.
	Serial Number	Indicates the manufacturer's serial number so that a specific medical device can be identified.
	Batch Code	Indicates the manufacturer's batch code so that the batch or lot can be identified.
	Unique Device Identifier	Indicates a carrier that contains unique device identifier information.
	European Conformity	European conformity (CE) mark for Class I medical devices.
	UK Conformity	UK Conformity (UKCA) mark for Class I medical devices (non-sterile, no measuring function, and not reusable surgical instrument).
	Canadian Standards Association	Indicates the device has been tested against applicable North American standards requirements.
	Authorized Representative in the European Community	Indicates the authorized representative in the European Community.
	Authorized Representative in United Kingdom	Indicates the authorized representative based in United Kingdom.
	Swiss Authorized Representative and Importer	Indicates the authorized representative and importer based in Switzerland.
	Distributor	Indicates the entity distributing the medical device into the local.
	Website address	Indicates the company website address.
	Waste Stream Disposal Status	Do not dispose of electronic products in the general wastestream.
	Caution	Indicates that caution is necessary when operating the device or control close to where the symbol is placed, or that the current situation needs operator awareness or operator action in order to avoid undesirable consequences

Symbol Glossary

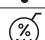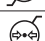Symbol	Symbol Title	Explanatory Text
	Telephone Receiver	Indicates the telephone contact number.
	Email address	Indicates the email contact address.
	Consult Instructions for Use	Indicates the need for the user to consult the instructions for use.
	Translation	Indicates the original medical device information has undergone a translation which supplements or replaces the original information.
	Manufacturer	Indicates the medical device manufacturer.
	Date of Manufacture	Indicates the date when the medical device was manufactured.
	Country of Manufacture	To identify the country of manufacture of products.
	Temperature Limit	Indicates the temperature limits to which the medical device can be safely exposed.
	Humidity Limitation	Indicates the range of humidity to which the medical device can be safely exposed.
	Atmospheric Pressure Limitation	Indicates the range of atmospheric pressure to which the medical device can be safely exposed.

Full explanation of symbols as used on Kerr packaging located at: <http://www.kerrdental.com/symbols-glossary>

INSTRUCTIONS FOR USE AND CARE

Orascoptic™ Spark™ Series Headlight

Specifications						
	Spark (Universal Mount)			Spark (Through-the-lens Mount)		
	Spark Standard	Spark TruColor	Spark TruColor (Generation 2)	Spark Standard	Spark TruColor	Spark TruColor (Generation 2)
Light Output	32 Lumens	32 Lumens	40 Lumens	32 Lumens	32 Lumens	40 Lumens
Color Rendering Index (CRI)	CRI 70+	CRI 90+	CRI 92+	CRI 70+	CRI 90+	CRI 92+
Correlated Color Temperature (CCT)	CCT 6500K	CCT 5700K	CCT 6500K	CCT 6500K	CCT 5700K	CCT 6500K
Weight of headlight with filter	.56 ounces (15.8 grams)			.43 ounces (12.2 grams)		
Diameter of headlight	.6 inches / 14 mm					
Weight of battery pack	.63 ounces (17.9 grams)					
Dimensions of battery pack	Length = 1.7 inches (43 mm) Width = 1.1 inches (28 mm) Height = 0.5 inches (13 mm)					
Charge time (each battery pack)	3 Hours					
Run time (each battery pack)	3 Hours					
Battery type	Lithium-ion / 3.7v / 425 mAh					
Electrical	Class II					
Operation mode	Continuous					
Standards	EMC = IEC60601-1-2, class B, EN55011 Safety = IEC60601-1, IEC 62471					
Operating environment	32° to 77°F (0° to 25°C) 10% to 85% relative humidity (non-condensing) 0.5 atm to 1.0 atm (500 hPa to 1060 hPa) atmospheric pressure					

Electromagnetic Compatibility Test	
CISPR 11 – Emissions: Radiated RF	Group 1, Class B
IEC 61000-4-2 – Immunity: Electrostatic Discharge	±8 kV contact ±15 kV air
IEC 61000-4-3 – Immunity: Radiated RF	3 V/m, 80 MHz – 2.7 GHz, 80 % 1 kHz AM
IEC 61000-4-3 – Immunity: Proximity Fields from RF Wireless Communications Equipment	IEC 60601-1-2:2014, section 8.10
IEC 61000-4-8 – Immunity: Power Frequency Magnetic Fields	30 A/m, 50/60 Hz

Troubleshooting	
Problem	Solution
Headlight does not turn on.	Check that the battery pack is charged. Remove and reattach the battery pack, making sure both locking tabs are secure. Attach the other battery pack to determine if the problem is in the headlight or battery pack.
Battery indicator light is blinking scarlet.	Indicates the battery is low. Charge the battery.
Battery indicator light does not illuminate amber while charging.	Check that the Micro-USB plug is secured in the charging port on the back of the battery pack. Check that the USB plug is secured in the power adapter. Check that the electrical outlet is producing power.

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Metrex Research, LLC dba Orasoptic
Kerr Italia S.r.l.
Nobel Biocare Services AG
Authorised Rep in UK: Kerr UK Ltd.