

Score<sup>S</sup>  
experts in seating

APOSTOLIDES  
SINCE 1929  
HEALTHCARE TECHNOLOGIES  
...your best companion!



ready-  
sit.  
go!

① score  
② ③ apical

instantly effective, sits comfortably, easy to operate

# ready. sit. go!

As a dental care specialist, you spend most of your working day sitting down. The Score Apical has been specially developed in cooperation with specialists and ergonomists to give you the right support to carry out your activities in a healthy working position. Even after a long day's work, you will still feel great! Less neck, shoulder and back pain. We've got your back.



## functional design - ready

With all its simplicity, the Score Apical is designed to be functional. The design is ergonomically tailored to the healthy sitting posture for dental care specialists.



## healthy posture - sit


The Score Apical encourages you to sit in the ergonomic S-position, minimising back, neck and shoulder complaints.



## easy to assemble - go

The Score Apical consists of three parts which you can assemble in seconds: the star base with castors, the gas lift and the compact seat with pelvic support. All parts are also separately replaceable to be as durable as possible.

### available in 12 stamskin colours:

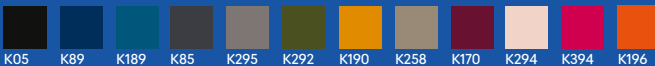

Prefer the Score Apical in the same upholstery as the dental treatment unit of the practice? Ask about the possibilities.



# set up

quick and easy

The Score Apical has several adjustment options to personalise it to your body and is simple and self-explanatory. To make the most of the Score Apical as an aid during your work, it is essential to adjust the settings to your body. We are happy to explain how you can do this.

To adjust your chair properly, the pelvic support must first be fully reclined, this can be done with the rear lever. Take a seat on the chair and make sure you sit firmly in the seat and that you do not sit too far back.

## ① seat height adjustment

Adjust the seat height with the front handle to an 110° knee angle.

## ② adjusting depth of pelvic support

Adjust the depth of the pelvic support with the rear handle and ensure it makes only slight contact to the pelvis.

## ③ adjusting the height of the pelvic support

Adjust the height of the pelvic support with the knob that is located on the backside of the support. The pelvic support should make contact at waistband height.



*Take a seat.*



# the right sitting posture

if you sit, sit properly!

The Score Apical is a chair that we developed together with ergonomists conforms to ISO standards 7493 Dentistry - Operator's stool and 11226 Static working postures. The result is a work chair that helps you automatically adopt an ergonomic sitting posture for your working posture so you are less likely to suffer from neck, shoulder and back complaints.



## ① head

To prevent neck complaints, the advice is to work with your head upright as much as possible; bend down a maximum of 20°. Therefore, make sure your chair is not too high!

## ② upper and lower arms

Place your elbows and upper arms relaxed against your upper body wherever possible to reduce strain. It is best to maintain an elbow angle of around 90°.

## ③ pelvis

Make sure you sit properly and upright in the seat. Do not lean against the pelvic support: make sure it only makes light contact with your pelvis. This will ensure your spine to stay in the S position.

## ④ legs

Sit symmetrically upright on the chair with your feet flat on the floor and your legs slightly spread in a 45° angle. The knees should be in a 110° angle. This posture provides you with good blood flow and therefore is much less straining for your body in the longer term.

# wide range

## dentel seating solutions from Score

With the introduction of the Score Apical, we complete our range for those who prefer to sit ergonomically on a work chair rather than a saddle stool. So, we have something for everyone. One thing they all have in common: they are ideal for the dental market!

With their star base and great adjustability, they are perfect for working in small spaces, regardless of your height. We have listed the various specifications for you below.

			
	<b>score apical</b>	<b>score balance alba</b>	<b>score amazone / jumper</b>
<b>Seating</b>	W 43 x D 34 cm	W 34 x D 30 cm / W 44 x D 30 cm	W 34 x D 30 cm / W 44 x D 30 cm
<b>Mechanism</b>	Fixed	Balance 360° Dynamic*	Fixed, Tilt* or Balance 360° Dynamic*
<b>Height adjustments +/- 3cm extra for balance mechanism</b>	Low MID25: 44 - 58 cm Standard MID31: 52 - 72 cm High MID37: 59 - 85 cm	Low MID25: 49 - 63 cm Standard MID31: 59 - 77 cm High MID38: 63 - 88 cm	Low MID25: 49 - 63 cm Standard MID31: 59 - 77 cm High MID38: 63 - 88 cm
<b>Made for people with the following heights:</b>	low MID25: 150 - 170 cm Standard MID31: 160 - 200 cm High MID37: > 185 cm	Low MID25: < 160 Standard MID31: 155 - 180 cm High MID38: > 165 cm	Low MID25: < 160 Standard MID31: 155 - 180 cm High MID38: > 165 cm
<b>Back support</b>	Pelvic support (standard)	-	Lumbar support (optional)
<b>Arm support</b>	-	360° support (optional)	360° support (optional)
<b>Bases</b>	Aluminium star base ø 54 cm or white star base ø 54 cm	Aluminium star base ø 54 cm or white star base ø 54 cm	Aluminium star base ø 50, ø 54, ø 60, ø 67 cm (with 360° support) or black / grey star base ø 54 cm
<b>Castors</b>	Soft ø 65 mm white, suitable for hard floors	Soft ø 65 mm white, suitable for hard floors	Soft ø 65 mm black, suitable for hard floors
<b>Upholstery</b>	Stamskin (seamless) in 12 colours	Stamskin in 12 colours	Stamskin (Bicolor) in 12 colours

\*Tilt - tilts the seat forwards and backwards \*Balance 360° Dynamic - allows your seat subtly move with your movements

# score

experts in seating

Our goal is to provide you with healthy and comfortable seating so you can feel fit and energized. Score has been an expert in ergonomic seating solutions for over 45 years, making a difference in the way you work. More than 100,000 oral healthcare professionals have improved their well-being and work experience with our seating solutions. We understand that movement is essential, but we also believe that sitting should be healthy.

We adhere to the latest ergonomic standards, employ cutting-edge technologies, and use premium materials, all while considering the unique challenges faced by oral healthcare professionals. At Score, we like to do things our way. We don't follow the crowd, but chart our course independently. You won't find chairs at Score that merely conform to trends; instead, you'll discover durable chairs designed to last for decades. We listen to our customers, and when we identify opportunities for improvement, we collaborate to bring them to life. Over the years, this approach has made us a trusted player in the market. Our aim is to ensure that you can perform your work with energy and relaxation, focusing entirely on your patients.

**APOSTOLIDES**  
SINCE 1929  
HEALTHCARE TECHNOLOGIES  
...your best companion!

## Αθήνα

Φειδιππίδου 23, Αθήνα  
210 6984 882

## Θεσσαλονίκη

Νεστ. Τέλλογλου 11  
2310 201 301

info@apostolides.gr - www.apostolides.gr

Scan the  
QR-code!



**Score**<sup>S</sup>  
experts in seating

📍 Feithspark 5  
9356 BX Tolbert  
The Netherlands

☎ +31 (0) 594 55 40 00  
✉ info@scoreseating.com  
🌐 scoreseating.com